

Vyafuser (AbbVie Infusion Pump) and Vyalev (foslevodopa/foscarbidopa solution)

The patient and support person know this medication and the pump best.

The medication and infusion pump are used for symptom management of Parkinson's disease.

If this is a medical emergency, stabilize the patient before addressing the pump.



Vyalev (foslevodopa / foscarbidopa) should remain infusing, except in the following situations:

- Suspected cellulitis related to the medication infusion.
- Patient is no longer able to maintain infusion (i.e. due to loss of consciousness).
- MRI (the pump is not MRI compatible, while the infusion set is. The infusion must be stopped and disconnected prior to the scan.)

Once the pump is stopped, oral medication must be started within ONE (1) hour.

If the patient or support person is not able, **STOP the pump** using the following instructions:

1. On the main STATUS SCREEN, press the MENU BUTTON
2. In the MENU SCREEN, use the UP ARROW (a) to highlight STOP PUMP.
3. Once STOP PUMP is highlighted, press SELECT, then YES.

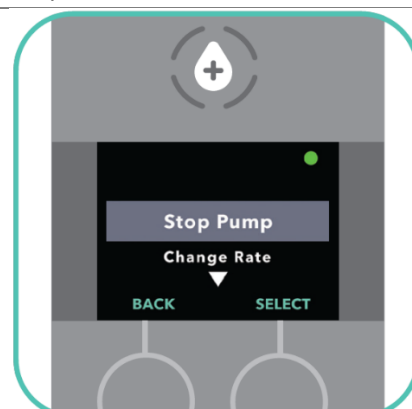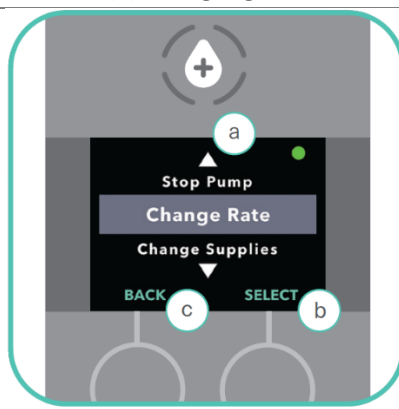

Additional instructions for management, including (re)initiating oral medication is available via:

- **Movement Disorders Clinic** – Monday to Friday 08:00 to 16:00. Ph: 403-944-4364, option 4
- **On-Call Neurologist:** ROCA: Area: Clinical Neurosciences, **Service: Neurology New Consults / Outside Calls**
- **Connect Care – Exceptional Care Plan** (visible as Patient FYI Flag)

